


# Thopaz™ Quick Card

 - This Quick Card in no way replaces the Instruction for Use REF 200.0685

## Prepare Thopaz for use

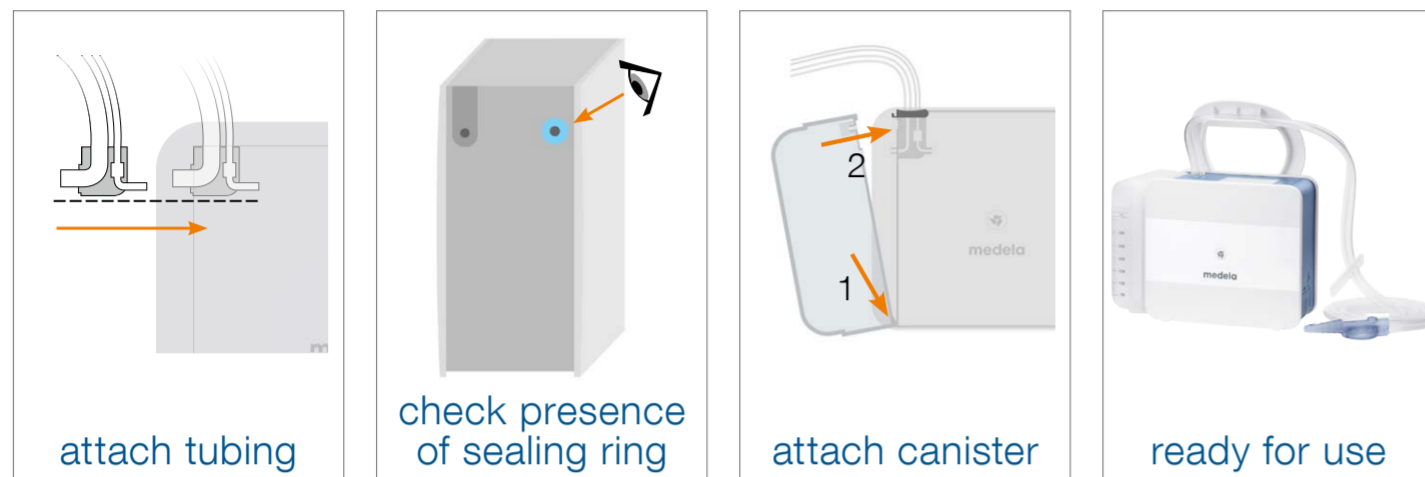

attach tubing

check presence of sealing ring

attach canister

ready for use

## Turn Thopaz on



press power button


Choose patient therapy number

keep number      new number

Standby-Mode

therapy number      switch on pressure

## Perform functional check

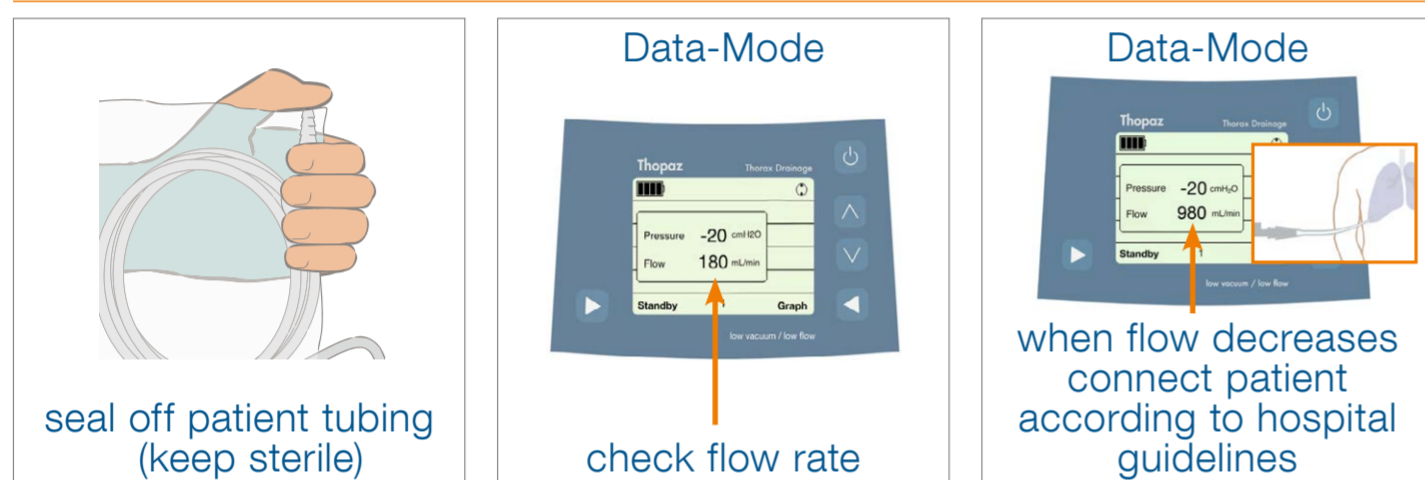

seal off patient tubing (keep sterile)


Data-Mode

check flow rate

Data-Mode

when flow decreases connect patient according to hospital guidelines

## Adjust pressure (when pump is running)

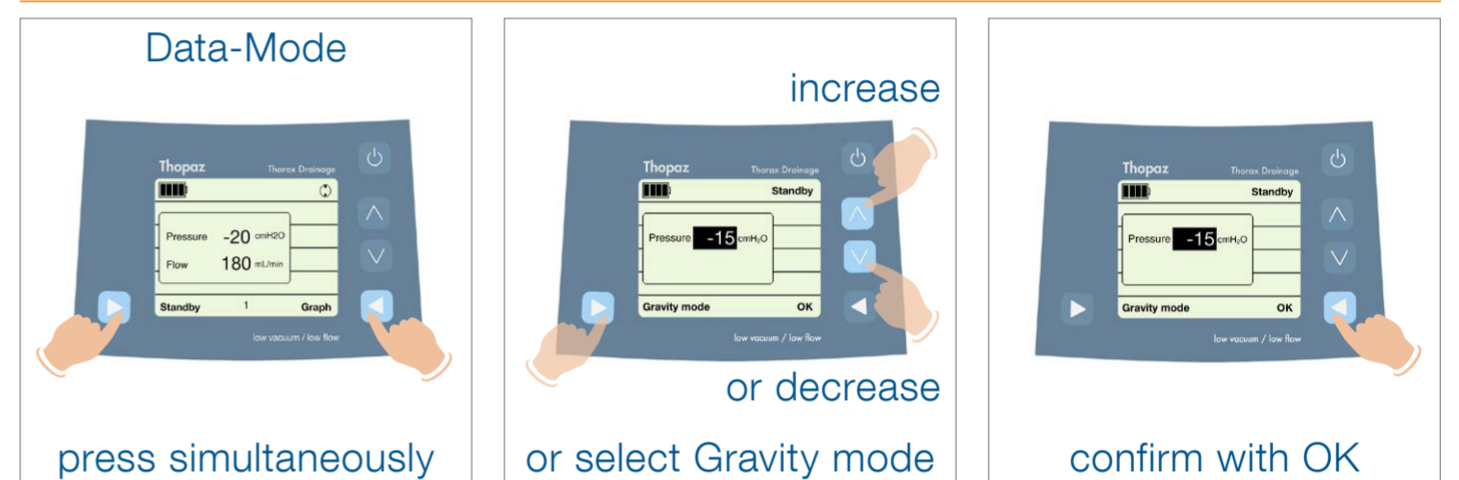

Data-Mode

press simultaneously

increase

or decrease or select Gravity mode

Data-Mode

confirm with OK

## Change canister



clamp of tubing

Data-Mode

press longer than 3 seconds

Thopaz in Standby-Mode

change canister

Standby-Mode


seal canister and dispose

Standby-Mode

press on to leave standby-mode

release clamp

## Check therapy progress

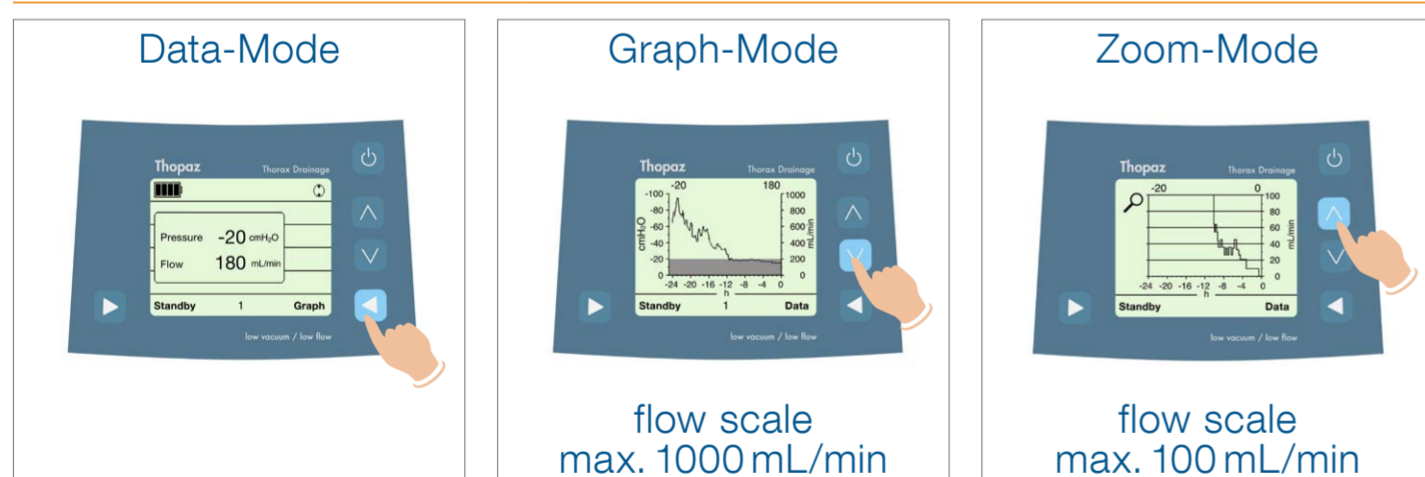

Data-Mode


Graph-Mode

flow scale max. 1000 mL/min

Zoom-Mode

flow scale max. 100 mL/min

## Catheter check

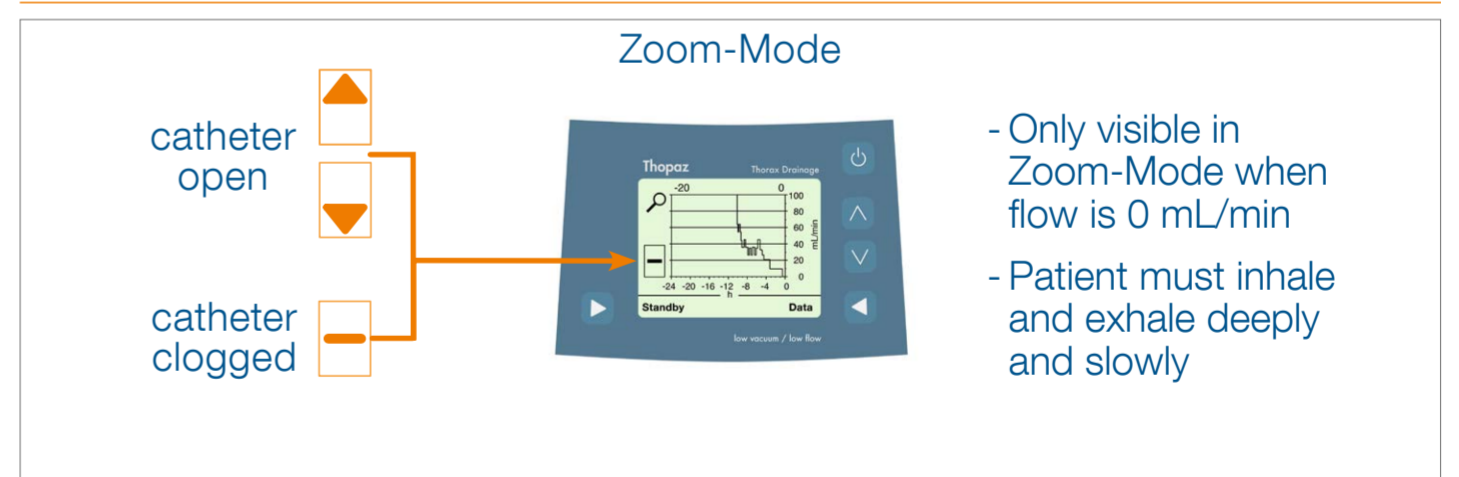

Zoom-Mode

catheter open

catheter clogged

- Only visible in Zoom-Mode when flow is 0 mL/min

- Patient must inhale and exhale deeply and slowly

## Alarm management



BEEP


error number

BEEP

press simultaneously

follow advice on screen

## Switch off Thopaz

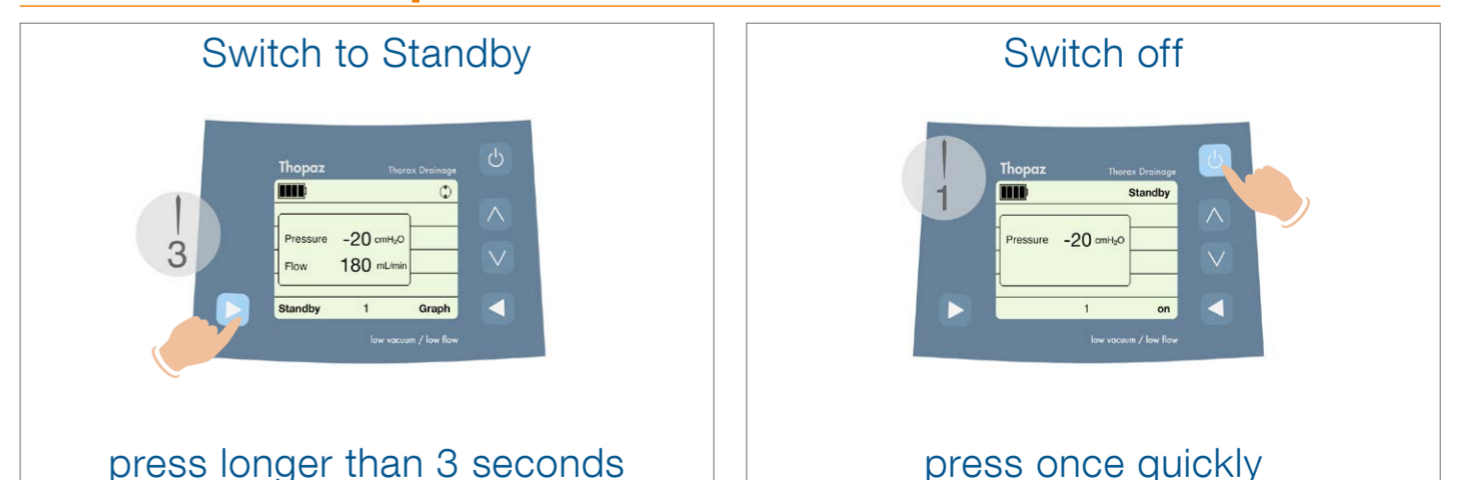

Switch to Standby

press longer than 3 seconds

Switch off

press once quickly